



Recording an Allergen Location Test

Recording an Allergen Location Test


1. Open the Allergen app.

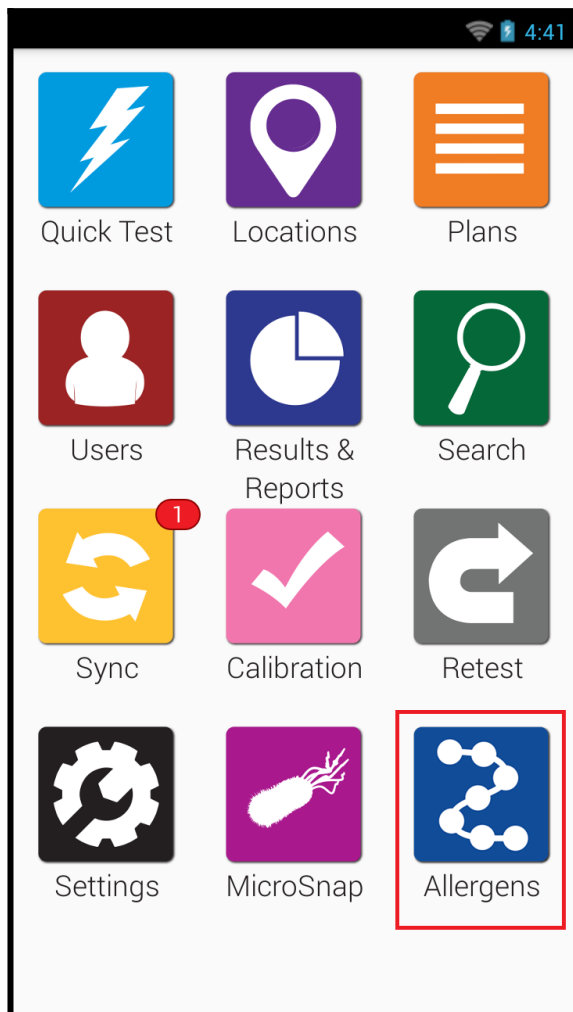

2. Tap Locations.

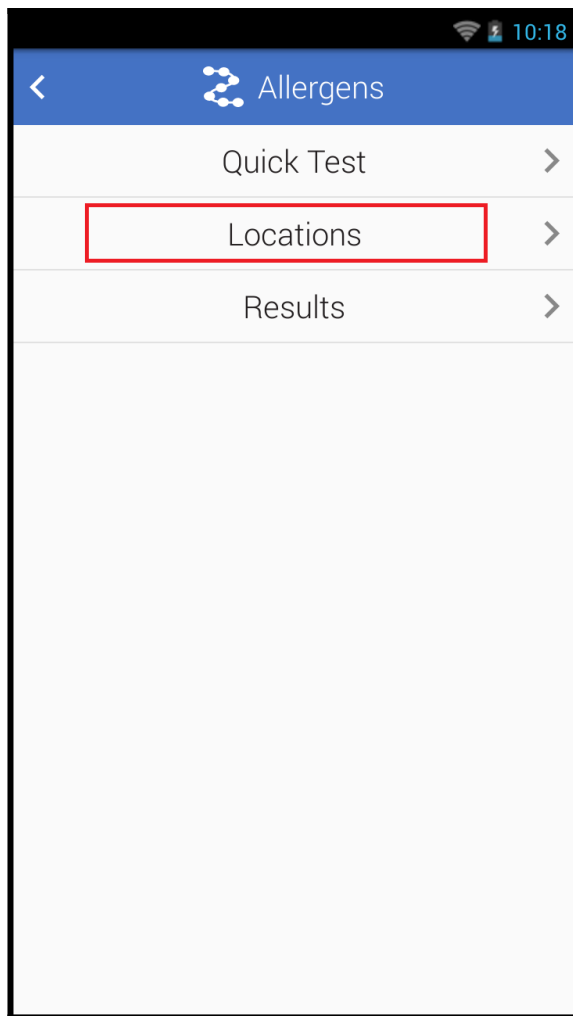

3. Tap the name of the location you are testing.

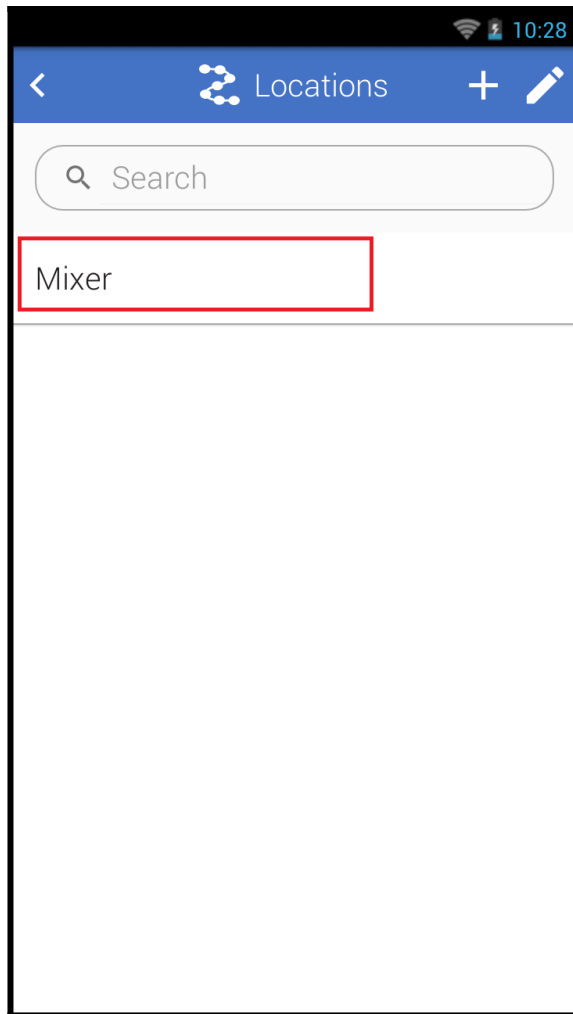

4. Confirm the device type and allergen are correct.
5. Enter the test result by tapping Positive, Negative, Caution, or Invalid.

10:32







< Location Test

Mixer

Device Type AlerTox Sticks

Allergens Peanut

Select the result that matched your Device

	Positive	Negative	Invalid
Control			
Positive			

6. Tap the Save button.
7. Review the test information and tap the Done button to finish recording the result.